
BTS 310[®]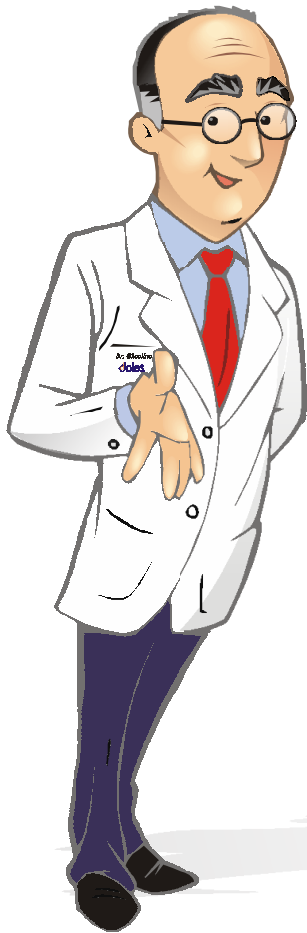

ÍNDICE

KIT	PÁGINA	KIT	PÁGINA
ALBUMINA	01	FOSFATASES	23
ALDOLASE	02	FOSFATO	24
ALT/TGP	03	FOSFATO UV	25
AMILASE	04	GAMAGT	26
AMILASE CNPG	05	GAMA GT CINÉTICA	27
AST/TGO	06	GLICOSE ENZ. LÍQUIDA	28
BILIRRUBINA	07	GLUCOX 500	29
CÁLCIO	08	HEMOGLOBINA	30
CÁLCIO ARSENAZO	09	HEMOGLOBINA GLICADA	31
CKMB (LANÇAMENTO)	10	MAGNÉSIO	32
CK NAC	11	MICROPROTE pirogalol	33
CLORETO COLORIMÉTRICO	12	MUCOPROTEÍNAS	34
COLESTEROL ENZ. LÍQUIDO	13	POTÁSSIO	35
COLESTEROL HDL	14	PROTEÍNAS TOTAIS	36
COLESTEROL 250.....	15	TRANSAMINASES	37
COLINESTERASE	16	TRIGLICÉRIDES 120	38
CPK.....	17	TRIGLICÉRIDES ENZ. LÍQUIDO	39
CREATININA CINÉTICA	18	URATO 160	40
DHL	19	URATO ENZIMÁTICO LÍQUIDO	41
DHL UV	20	UREIA 500	42
FERRO	21	UREIA UV	43
FOSFATASE ALCALINA CINÉTICA	22		

ALBUMINA

END POINT WITH STANDARD

UNITS:	g/dL
READING MODE:	MONOCHROMATIC
READING FILTER:	670
STABILIZ. TIME:	3
TEMPERATURE:	37
SIPPING VOLUME:	900
REACTION SLOPE:	INCREASING
STANDARD:	4.000

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ALT/TGP

DELTA KINETICS WITH FACTOR

UNIT:	UI/L
READING FILTER:	340nm
FACTOR:	1746
STABILIZ. TIME:	1
INCUBATION TIME:	60
INTERVAL TIME:	20
INTERVALS NUMBER:	4
SIPPING VOLUME:	400
REACTION SLOPE:	Decrease
TEMPERATURE:	37

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AMILASE CNPG

DELTA KINETICS WITH FACTOR

UNIT:	UI/L
READING FILTER:	405nm
FACTOR:	3953
STABILIZ. TIME:	1
INCUBATION TIME:	30
INTERVAL TIME:	60
INTERVALS NUMBER:	2
SIPPING VOLUME:	500
REACTION SLOPE:	Increase
TEMPERATURE:	37

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CÁLCIO

END POINT WITH STANDARD

UNITS:	mg/dL
READING MODE:	MONOCHROMATIC
READING FILTER:	578
STABILIZ. TIME:	3
TEMPERATURE:	37
SIPPING VOLUME:	*
REACTION SLOPE:	INCREASING
STANDARD:	10.00

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CÁLCIO ARSENAZO

END POINT WITH STANDARD

UNITS:	mg/dL
READING MODE:	MONOCHROMATIC
READING FILTER:	670
STABILIZ. TIME:	3
TEMPERATURE:	37
SIPPING VOLUME:	500
REACTION SLOPE:	INCREASING
STANDARD:	10

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CK MB

FIXED TIME WITH FACTOR

UNIT:	U./L
READING FILTER:	340nm
FACTOR:	1333
STABILIZ. TIME:	1
INCUBATION TIME:	180
INTERVAL TIME:	300
SIPPING VOLUME:	500
REACTION SLOPE:	Increase
TEMPERATURE:	37

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CK-NAC

DELTA KINETICS WITH FACTOR

UNIT:	U/L
READING FILTER:	340nm
FACTOR:	8199
STABILIZ. TIME:	1
INCUBATION TIME:	120
INTERVAL TIME:	60
INTERVALS NUMBER:	3
SIPPING VOLUME:	400
REACTION SLOPE:	Increase
TEMPERATURE:	37

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CLORETOS COLORIMÉTRICO

END POINT WITH STANDARD

UNITS:	mEq/L
READING MODE:	MONOCHROMATIC
READING FILTER:	505
STABILIZ. TIME:	3
TEMPERATURE:	37
SIPPING VOLUME:	900
REACTION SLOPE:	INCREASING
STANDARD:	100.0

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COLESTEROL ENZ. LÍQUIDO

END POINT WITH STANDARD

UNITS:	mg/dL
READING MODE:	MONOCHROMATIC
READING FILTER:	505
STABILIZ. TIME:	3
TEMPERATURE:	37
SIPPING VOLUME:	900
REACTION SLOPE:	INCREASING
STANDARD:	200.0

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COLESTEROL HDL

END POINT WITH STANDARD

UNITS:	mg/dLS
READING MODE:	MONOCHROMATIC
READING FILTER:	505
STABILIZ. TIME:	3
TEMPERATURE:	37
SIPPING VOLUME:	900
REACTION SLOPE:	INCREASING
STANDARD:	100.0

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CREATININA CINÉTICA

FIXED TIME WITH STANDARD

UNIT:	mg/dL
READING FILTER:	505nm
STABILIZ. TIME:	1
INCUBATION TIME:	30
INTERVAL TIME:	60
SIPPING VOLUME:	500
REACTION SLOPE:	Increase
STANDARD:	5
TEMPERATURE:	37

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DHL - UV

DELTA KINETICS WITH FACTOR

UNIT:	UI/L
READING FILTER:	340nm
FACTOR:	8109
STABILIZ. TIME:	1
INCUBATION TIME:	30
INTERVAL TIME:	60
INTERVALS NUMBER:	3
SIPPING VOLUME:	400
REACTION SLOPE:	Decrease
TEMPERATURE:	37

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FOSFATASE ALCALINA CINÉTICA

DELTA KINETICS WITH FACTOR

UNIT:	UI/L
READING FILTER:	405nm
FACTOR:	2764
STABILIZ. TIME:	1
INCUBATION TIME:	60
INTERVAL TIME:	60
INTERVALS NUMBER:	3
SIPPING VOLUME:	400
REACTION SLOPE:	Increase
TEMPERATURE:	37

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FOSFATO

END POINT WITH STANDARD

UNITS:	mg/dL
READING MODE:	MONOCHROMATIC
READING FILTER:	670
STABILIZ. TIME:	3
TEMPERATURE:	37
SIPPING VOLUME:	900
REACTION SLOPE:	INCREASING
STANDARD:	5.000

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FOSFATO - UV

END POINT WITH STANDARD

UNITS:	mg/dL
READING MODE:	MONOCHROMATIC
READING FILTER:	340
STABILIZ. TIME:	3
TEMPERATURE:	37
SIPPING VOLUME:	900
REACTION SLOPE:	INCREASING
STANDARD:	5.000

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GAMA GT CINÉTICA

DELTA KINETICS WITH FACTOR

UNIT:	U/L
READING FILTER:	405nm
FACTOR:	1111
STABILIZ. TIME:	1
INCUBATION TIME:	20
INTERVAL TIME:	60
INTERVALS NUMBER:	3
SIPPING VOLUME:	400
REACTION SLOPE:	Increase
TEMPERATURE:	37

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GLICOSE ENZ. LÍQUIDA

END POINT WITH STANDARD

UNITS:	mg/dL
READING MODE:	MONOCHROMATIC
READING FILTER:	505
STABILIZ. TIME:	3
TEMPERATURE:	37
SIPPING VOLUME:	900
REACTION SLOPE:	INCREASING
STANDARD:	100.0

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MAGNÉSIO

END POINT WITH STANDARD

UNITS:	mg/dL
READING MODE:	MONOCHROMATIC
READING FILTER:	505
STABILIZ. TIME:	3
TEMPERATURE:	37
SIPPING VOLUME:	900
REACTION SLOPE:	INCREASING
STANDARD:	2.000

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MICROPROTE pirogalol

END POINT WITH STANDARD

UNITS:	mg/dL
READING MODE:	MONOCHROMATIC
READING FILTER:	578
STABILIZ. TIME:	3
TEMPERATURE:	37
SIPPING VOLUME:	500
REACTION SLOPE:	INCREASING
STANDARD:	100

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MUCOPROTEÍNAS

END POINT WITH STANDARD

UNITS:	mg/dL
READING MODE:	MONOCHROMATIC
READING FILTER:	670
STABILIZ. TIME:	3
TEMPERATURE:	37
SIPPING VOLUME:	900
REACTION SLOPE:	INCREASING
STANDARD:	6.000

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POTÁSSIO

END POINT WITH STANDARD

UNITS:	mEq/L
READING MODE:	MONOCHROMATIC
READING FILTER:	578
STABILIZ. TIME:	3
TEMPERATURE:	37
SIPPING VOLUME:	900
REACTION SLOPE:	INCREASING
STANDARD:	5.000

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PROTEÍNAS TOTAIS

END POINT WITH STANDARD

UNITS:	g/dL
READING MODE:	MONOCHROMATIC
READING FILTER:	546
STABILIZ. TIME:	3
TEMPERATURE:	37
SIPPING VOLUME:	900
REACTION SLOPE:	INCREASING
STANDARD:	4.000

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.TGO/AST

MULTISTANDARD SPLINE

UNITS: F.R./mL
READING MODE: MONOCHROMATIC
READING FILTER: 505
STABILIZ. TIME: 3
TEMPERATURE: 37
SIPPING VOLUME: 900
REACTION SLOPE: INCREASING
REPLICATES STD.: 1
REPLICATES SAMP.: 1

AXIS

ABS: LIN
CONC: LIN
STANDARD 1: 0.000
STANDARD 2: 24.00
STANDARD 3: 61.00
STANDARD 4: 114.0
STANDARD 5: 190.0
DILUTION FACTOR: 1.000

.TGP/ALT

MULTISTANDARD SPLINE

UNITS: F.R./mL
READING MODE: MONOCHROMATIC
READING FILTER: 505
STABILIZ. TIME: 3
TEMPERATURE: 37
SIPPING VOLUME: 900
REACTION SLOPE: INCREASING
REPLICATES STD.: 1
REPLICATES SAMP.: 1

AXIS

ABS: LIN
CONC: LIN
STANDARD 1: 0.000
STANDARD 2: 28.00
STANDARD 3: 57.00
STANDARD 4: 97.00
STANDARD 5: 150.0
DILUTION FACTOR: 1.000

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.TGO/AST

MULTISTANDARD SPLINE

UNITS: U.I./L
READING MODE: MONOCHROMATIC
READING FILTER: 505
STABILIZ. TIME: 3
TEMPERATURE: 37

SIPPING VOLUME: 900
REACTION SLOPE: INCREASING
REPLICATES STD.: 1
REPLICATES SAMP.: 1
AXIS
ABS: LIN
CONC: LIN
STANDARD 1: 0.000
STANDARD 2: 11.50
STANDARD 3: 29.40
STANDARD 4: 54.94
STANDARD 5: 91.58
DILUTION FACTOR: 1.000

.TGP/ALT

MULTISTANDARD SPLINE

UNITS: U.I./L
READING MODE: MONOCHROMATIC
READING FILTER: 505
STABILIZ. TIME: 3
TEMPERATURE: 37
SIPPING VOLUME: 900
REACTION SLOPE: INCREASING
REPLICATES STD.: 1
REPLICATES SAMP.: 1

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URATO 160

END POINT WITH STANDARD

UNITS:	mg/dL
READING MODE:	MONOCHROMATIC
READING FILTER:	505
STABILIZ. TIME:	3
TEMPERATURE:	37
SIPPING VOLUME:	900
REACTION SLOPE:	INCREASING
STANDARD:	7.000

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UREIA 500

END POINT WITH STANDARD

UNITS:	mg/dL
READING MODE:	MONOCHROMATIC
READING FILTER:	670
STABILIZ. TIME:	3
TEMPERATURE:	37
SIPPING VOLUME:	900
REACTION SLOPE:	INCREASING
STANDARD:	80.00

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UREIA-UV

FIXED TIME WITH STANDARD

UNIT:	mg/dL
READING FILTER:	340nm
STABILIZ. TIME:	1
INCUBATION TIME:	30
INTERVAL TIME:	60
SIPPING VOLUME:	400
REACTION SLOPE:	Decrease
STANDARD:	80
TEMPERATURE:	37

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